- CS - SMOKE EVACUATORS

CS200 - CS200LC - CS900LC

CS-SMOKE EVACUATORS

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CS-SMOKE EVACUATORS

The smoke evacuators **CS** are innovative suction units, with new characteristics, which have been designed to evacuate and filter the smoke produced during all the electro-surgical procedures, by eliminating the bad smells or the bacteriological risks, and by solving the very common problem of the visual field during the laparoscopic procedures.

The **CS200** is equipped with an electric suction group at high flow and medium noise; moreover it has got the function *TURBO - high suction*. It represents the basic model, which is ideal for the out-patient treatment, thanks to the connection with the speculum in gynaecology, the suction systems on the electrodes-handle, and the fixed tubing on the operation point.

The **CS900LC** - the most complete model - is equipped with an electric suction group at high flow and low noise, a special circuit for the connection to a centralized vacuum system, and another circuit for the controlled output of the CO₂ which is used in the laparoscopic procedures with the pneumoperitoneum. It is the ideal model for the usage in the operating theatre, thanks to the low level of noise produced, and the big range of the available procedures. It allows what follows:

- The evacuation of the smoke during the surgical open sky procedures, thanks to the aspiration system on the electrodes-handle or by an independent cannulae, through the suction group or the external vacuum system.
- The adjustable evacuation of the smoke during the laparoscopic procedures with the CO₂ pneumoperitoneum, thanks to the connection with the trocar, through the controlled output of the gas itself.
- The adjustable evacuation of the smoke during the gas-less laparoscopic procedures, thanks to the connection with the trocar, through the suction group or the external vacuum system.

The **CS200LC** is the same as the previous one, but it is equipped with a less powerful electric suction group. It is ideal for the suction during the laparoscopic procedures. It allows what follows:

- The evacuation of the smoke during the laparoscopic procedures with the CO₂ pneumoperitoneum, through the adjustable leakage of the gas itself.
- The evacuation of the smoke, with adjustable flow, during the gas-less laparoscopic procedures, through the suction group or the external vacuum system.

All the models can be used as a "shelf" for the electro surgical units; there are also specific trolleys, both for the out-patient treatment and the operating theatre.

GENERAL TECHNICAL CHARACTERISTICS

GENERAL TECHNICAL CHARACTERISTICS

- **Microprocessor Control System** - Functioning control through micro-processor, with auto-check at the switching-on.
- **Three Stages Filter** - Three stages filter: two antibacterial Hepa type, with a total efficiency 99,99999% for 0.3 micron particles, and one anti-smell with active carbons, controlled by a micro-processor with double check (filter efficacy, limit time of usage of 2 hours), luminous alarm signal and functioning block after the first switching-on before the intervention of the control circuit.
- **Automatic Start/Stop Sensing System** - Automatic switching on / switching off of the suction system, every time the electro surgical unit is activated, through a non-contact sensing system which eliminates the need of both specific connections to the electro surgical unit and a pedal switch (available just in case the operator prefers this option).
- **Programmable Stop Delay** - Adjusting of the delay in the switching off of the suction system from 0 to at least 30sec, in order to completely eliminate the smoke.
- **Switching on Pedal** - Pneumatic, waterproof, anti-explosion, for operating theatre IPX7/AP/APG (non standard, but available on request)
- **Enclosure** - Metallic and painted, it can be used as a "shelf" for the electro surgical unit, or it can be put in specific trolleys.
- **Cooling** - By convection, no fan.
- **Classification IEC 60601-1** - Class I, type BF (leakage to the enclosure: less than 25microA; leakage to the patient: absent).
- **Safety Rules** - The units meet IEC 601-1 Safety Rules.
- **Classification and Approval EC 93/42** - Class IIA - CE0051.
- **Dimensions and Weight** - 40x40x21cm, 17kg.

TECHNICAL CHARACTERISTICS CS200

- **Suction Group** - Rotative, non lubricated - Flow: 500lt/min; Max. vacuum: 110mmHg.
- **Suction Regulation** - Electronic, with function *Turbo - high suction*. Rotative control with visual bar-led
- **Mains / Absorption / Fuses / Switch** - 230V-50/60Hz / 650VA / 2x5A / Bipolar, green.

TECHNICAL CHARACTERISTICS CS900LC

- **Suction Group** - Rotative, lubricated - Flow:50lt/min; Max. vacuum: 720/730mmHg.
- **Suction Regulation** - Rotative control, with graduated scale.
- **Mains / Absorption / Fuses / Switch** - 230V-50/60Hz / 250VA / 2x1,6A / Bipolar, green.
- **Connection to the external vacuum system** - Connector with rapid device.
- **Control of the CO₂ output** - Circuit with pressure tight electro-valve.

TECHNICAL CHARACTERISTICS CS200LC

- **Suction Group** - Rotative, with membrane / piston - Flow: 23lt/min; Max. vacuum: 650/700mmHg.
- **Suction Regulation** - Rotative control, with graduated scale.
- **Mains / Absorption / Fuses / Switch** - 230V-50/60Hz / 138VA / 2x1A / Bipolar, green.
- **Connection to the external vacuum system** - Connector with rapid device.
- **Control of the CO₂ output** - Circuit with pressure tight electro-valve.

STANDARD ACCESSORIES

Model CS200

- 1 - Filter group (antibacterial Maxipleat/2, antibacterial Deas1911, anti-smell with active carbons Scavenger).
- 1 - Non sterile tubing **T19-1.5** (for a fixed suction system, or extension of **T11-1.8**).
- 1 - Disposable sterile tubing **T11-1.8** for the connection to the speculum, to the suction set for electrodes-handle, to the sterile handle with suction system.

Model CS900LC

- 1 - Filter group (antibacterial Maxipleat/2, antibacterial Deas1911, anti-smell with active carbons Scavenger).
- 1 - Non sterile tubing **T19-1.5** (for a fixed suction system, or extension of **T11-1.8**).
- 1 - Disposable sterile tubing **T11-1.8** for the connection to the speculum, to the suction set for the electrodes-handle, to the sterile handle with suction system and to the trocar in laparoscopy.
- 1 - Tubing TC/CV for the connection to the external vacuum systems.

Model CS200LC

- 1 - Filter group (antibacterial Maxipleat/2, antibacterial Deas1911, anti-smell with active carbons Scavenger).
- 1 - Non sterile tubing **T19-1.5** (for a fixed suction system, or extension of **T11-1.8**).
- 1 - Disposable sterile tubing **T11-1.8** for the connection to the speculum, to the suction set for the electrodes-handle, to the sterile handle with suction system and to the trocar in laparoscopy.
- 1 - Tubing TC/CV for the connection to the external vacuum systems.

GENERAL LIST OF ACCESSORIES

Internal Filters.

SCA - SCAVENGER, Anti-smell filter with active carbons. (*)

MAX2 - MAXIPLAET/2, Antibacterial/Antiviral filter. (*)

External Sterile Filters for suction tubing.

DE19 - DEAS1911, Sterile Antibacterial/Antiviral filter for suction tubing. (*)

Disposable sterile suction tubing.

T11-1.8 - Disposable sterile tubing, l. mt1,8 - Ø 11mm. (*)

Extension tubing for disposable sterile tubing T11 or suction tubing with GRA.

T19-1.5 - Extension tubing, l. mt1,5 - Ø 19mm. (*)

Suction tubing in autoclavable silicone.

TS/T-2.5 - Tubing, l. mt.2,5 and Ø 7x12 with connection to the unit.

Disposable sterile set for laparoscopy or general applications.

STERSET - Antibacterial filter with tubing 5mt. long and luer lock connection. (*)

Suction set for reusable and autoclavable electrodes-handles.

SES - Smoke Evacuation Sleeve - Suction set for disposable sterile electrodes-handles. (*)

Monopolar electrodes-handles with suction system and manual switches.

SY-PMOD-IIB ST - Disposable sterile handle with short blade electrode (shaft 5/6cm.), with adjustable suction system, and hand activation switches - international standard connection with three pins. (*)

SY-PMOD-IIB LT - See above, with long blade electrode (shaft 11/12cm.). (*)

Connection tubing to the external suction system (for mod. CS200LC or CS900LC).

TC/CV - Connection tubing l. mt5, completed with rapid connection to the unit.

It is not equipped with any connector for the centralized system, as we do not know the type of the connection which is used.

Trolley, mobile arm for the tubing, and rigid suction system.

H10 -Trolley with wheels and two shelves, cm. 40x40x75, wheels Ø 50mm.

H10/AB - See above, antistatic wheels, 2 with castors.

H25 -Trolley with wheels and two shelves **for the electro surgical unit and the smoke evacuator**, equipped with a container for the accessories and a shelf for the pedals. Antistatic wheels Ø 50mm, two with castors. Dimensions: cm50x60x100.

H26 - See above, **for the electro surgical unit, the smoke evacuator, and the gas Argon coagulator** (with seats for the Argon cylinders)

EV/A - Mobile arm for rigid suction system **GRA** (only for **H10** and **H10/AB**).

GRA - Rigid suction system (for out-patient treatment).

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- H10/AB** - Idem. Pero con ruedas antiestáticas, 2 con frenos.
- EV/A** - Brazo móvil para sistema rígido de aspiración **GRA** (solo para **H10** y **H10/AB**).